

Congratulations!

You upgraded to T-Scan[®] Novus[™]!

What's new with the Novus Handpiece?

Power On Indicator. Shows the Handpiece is powered on.

Sensor OK Indicator. Shows the sensor is correctly inserted, and a new scan can be opened.

Sensitivity +/- **Buttons**. Increase / decrease the sensitivity of the sensor.

Scan Mode Indicator. Shows the sensor is scanning force data and transferring data to the PC.

Record Button (New Scan / Start / Stop). Multi-function button. Used to open a new scan. Once open, the button can be used to start and stop recording a scan.





Handpiece Latch. When the latch is pushed in the "down" position, the latch retracts the contact pins inside the Handpiece to allow insertion of the sensor tab. When Pushed "up" the sensor is fully inserted and locked in place.

Non-slip Rubber feet. Stabilizes the Handpiece on an operatory tray.

Cable Access Cover. Unscrew to remove / replace the Handpiece Micro-B USB Cable.



CONSULT YOUR SOFTWARE HELP FILE FOR MORE INFORMATION ON THE NOVUS HANDPIECE.